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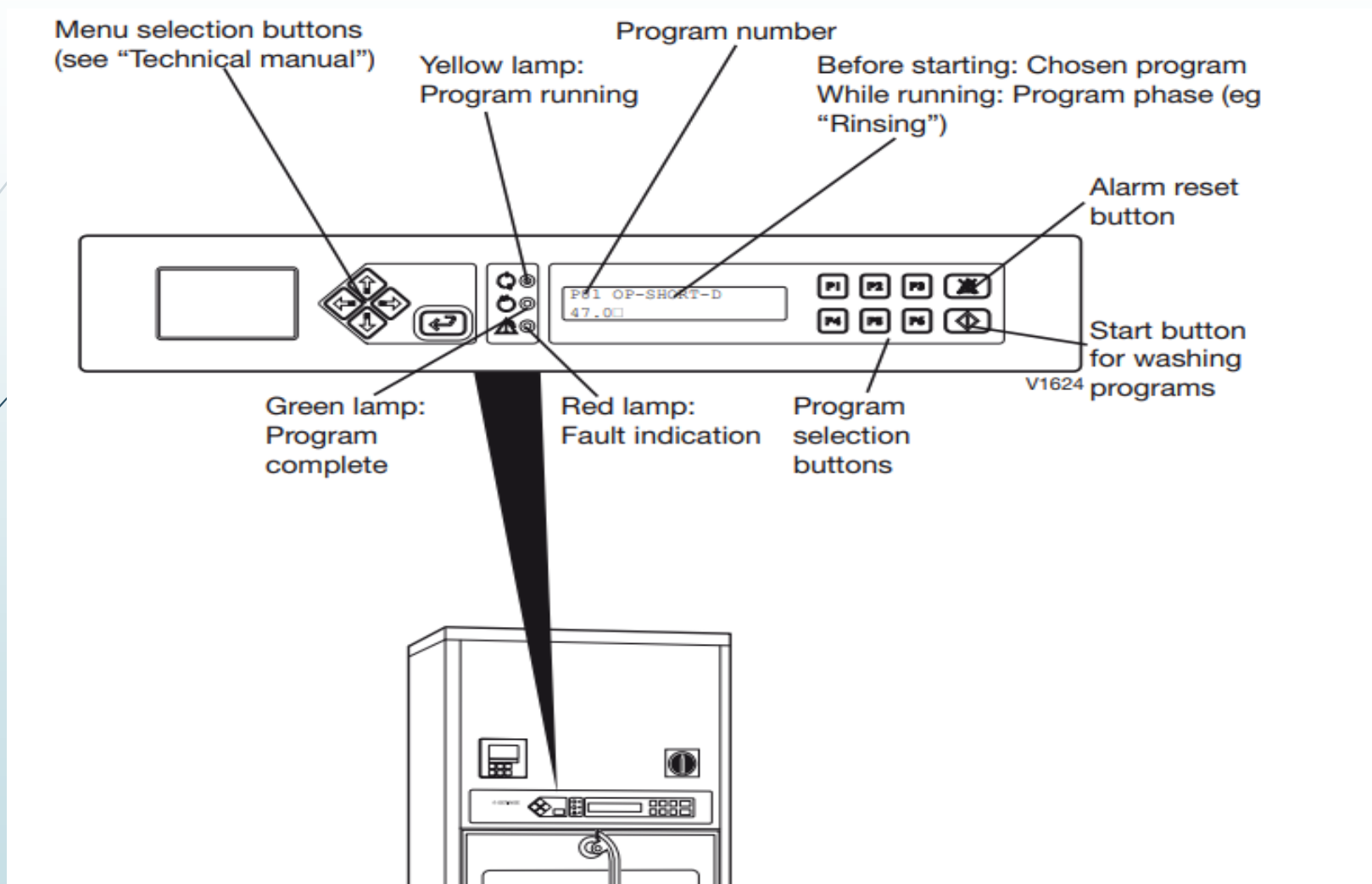
HOSPITAL A₀ 3000 (90 °C - 5 MIN)

This program sheet applies to 5029551, -21,-23,-24, -26, -27

Programs	Program selection button					Type of goods
5029551	-21	-23	-24	-26	-27	
POWER CLEAN	<input type="button" value="P1"/>	<input type="button" value="P1"/>	<input type="button" value="P1"/>	<input type="button" value="P1"/>	<input type="button" value="P1"/>	Very dirty instruments
OP-SHORT-D	<input type="button" value="P2"/>		<input type="button" value="P2"/>	<input type="button" value="P2"/>		Instruments, tubular instruments
OP-D	<input type="button" value="P3"/>		<input type="button" value="P3"/>	<input type="button" value="P3"/>		Instruments, tubular instruments
OP-LONG-D	<input type="button" value="P4"/>		<input type="button" value="P4"/>	<input type="button" value="P4"/>		Very dirty instruments
AN-D	<input type="button" value="P5"/>	<input type="button" value="P5"/>	<input type="button" value="P5"/>	<input type="button" value="P5"/>	<input type="button" value="P5"/>	Anaesthesia tubing
ENZYM-SHORT-D			P8*			Instruments, tubular instruments
ENZYM-D			P9*			Instruments, tubular instruments
OP-SHORT-D-3		<input type="button" value="P2"/>			<input type="button" value="P2"/>	Instruments, tubular instruments
OP-D-3		<input type="button" value="P3"/>			<input type="button" value="P3"/>	Instruments, tubular instruments
OP-LONG-D-3		<input type="button" value="P4"/>			<input type="button" value="P4"/>	Very dirty instruments
OP-BGA (A+B)-D-3		P7*			P7*	Circulation goods, instruments. (90 °C - 10 min).
OS-CHEM-D				P8*	P8*	OP-shoes
OP-BGA (A+B)-D	P7*		P7*	P7*		Circulation goods, instruments. (90 °C - 10 min).
AN-BGA (A+B)-D	<input type="button" value="P6"/>	<input type="button" value="P6"/>	<input type="button" value="P6"/>	<input type="button" value="P6"/>	<input type="button" value="P6"/>	Anaesthesia tubing. (90 °C - 10 min)
DRYING	P11*	P11*	P11*	P11*	P11*	Drying (Only available on machines with this option)
DESCALING	P12*	P12*	P12*	P12*	P12*	Descaling of chamber, inserts washer arms and washer vanes. Dose 100 ml Neudish BU (sold by Dr.Weigert) into the wash chamber. NOTE: Insert a empty washcart into the chamber when running this program. Do not have any goods in the machine with this program. It is only for descaling the machine.

* For choosing this program, see instructions for use.

Getinge detergent for mechanical washing of:

A - Surgical instruments

B - MIS-instruments

C - Anesthesia equipment

D - OP-shoes

E - Containers

F - Baby-bottles

G - Lab glassware

Products	Alkalinity	Surfactants	Phosphates	Enzymes	Suitable for optics and anodized Alu.	Goods to be washed
Getinge Power Wash	Amines	+	-	+	+	A, B, C, D, E
Getinge Wash Extra	Alkaline salts	-	+	-	(+)	A, C, D, (E), F
Getinge Wash Intensive	Caustic soda	+	-	-	-	A, C, D, F
Getinge Wash Enzyme	-	+	-	+	+	A, B, C, D, E
Getinge Wash LAB	Caustic soda	-	-	-	-	G



Acidic detergents and neutralizing agents

Products	Acids	Effects	Prewash for	Dosage lab glassware
Getinge Neutralization P	Phosphoric acid, Citric acid	Neutralization of alkaline residues, removal of mineral residues, e.g. rust and lime	Removal of mineral residues	First rinse after the alkaline wash for neutralization or prewash
Getinge Neutralization	Citric acid	Neutralization of alkaline residues		First rinse after the alkaline wash

Lubricants and rinse aids

Products	Ingredients	Effects	Dosage
Getinge Instrumentmilk	Paraffinum perliquidium emulsifying agents	Lubrication of instruments	Final rinse
Getinge Rinse	Surfactants	Reduction of surface tension of water, improved drying	Final rinse

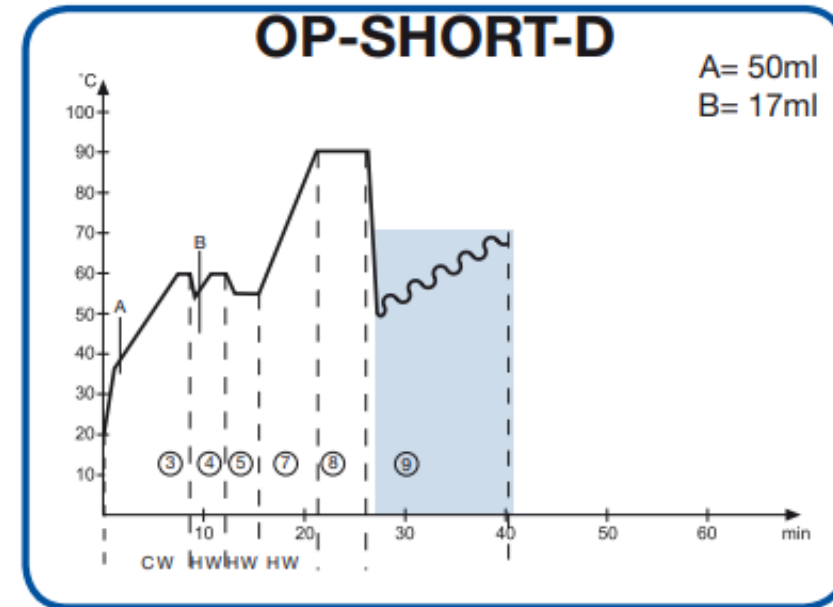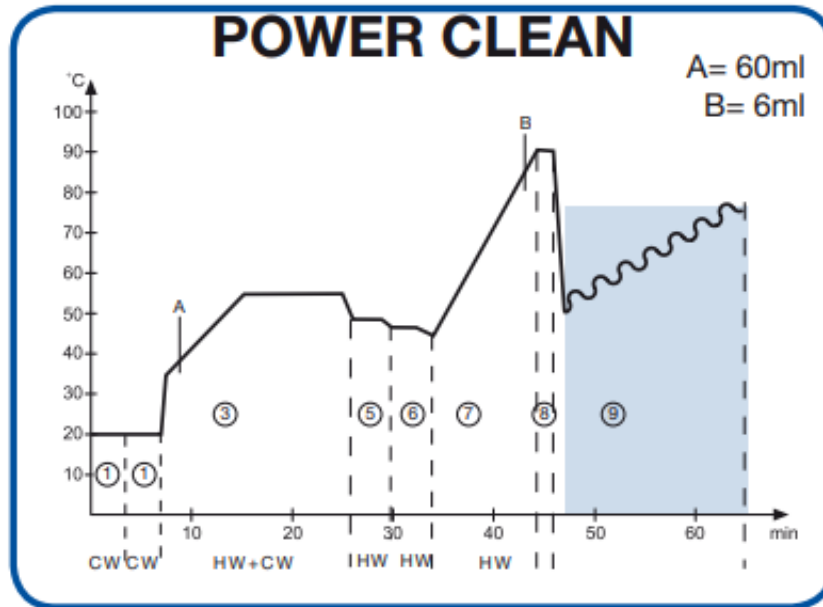GETINGE

Program phases

- | | | | |
|---|-----------------|---|---|
| 1 | Pre-rinse 1 | A | Alkaline detergent |
| 2 | Pre-rinse 2 | B | Rinse/aid |
| 3 | Wash | C | Instrument milk (option) When dosing instrument milk, rinse-aid is not dosed. |
| 4 | Neutralize | D | Enzym |
| 5 | Post-rinse 1 | E | Neutralizing |
| 6 | Post-rinse 2 | F | Chemical Disinfect. |
| 7 | Final rinse | | |
| 8 | Disinfection | | |
| 9 | Drying (Option) | | |

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
	Aluminum	Anodized alum	Titanium	Non ferrous m	Machrome pla	Stainless steel	Glass	Ceramics	Rubber	Plastic
 Getinge Clean Pre-treatment Foam	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
 Getinge Clean Manual Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
 Getinge Clean Enzymatic Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	M
 Getinge Clean Enzymatic Detergent	✓	✓	✓	✓	✓	✓	✓	✓	✓	M
 Getinge Clean MIS Detergent	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
 Getinge Clean Universal Detergent	M	M	M	M	✓	✓	✓	✓	✓	M
 Getinge Clean Heavy Soil Detergent	-	-	M	-	✓	✓	✓	✓	✓	M
 Getinge Clean Prion Detergent	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
 Getinge Clean Neutralizer	M	M	✓	✓	✓	✓	✓	✓	M	M
 Getinge Clean Neutralizer Plus	-	-	M	M	M	✓	✓	M	M	M
 Getinge Clean Instrument Lubricant	✓	✓	✓	✓	✓	✓	-	-	-	-
 Getinge Clean Instrument Lubricant Plus	✓	✓	✓	✓	✓	✓	-	-	-	-
 Getinge Clean Rinse Aid	✓	✓	✓	✓	✓	✓	✓	✓	M	M
 Getinge Clean Chamber Foam	-	-	-	-	-	✓	-	-	-	-

✓ = The product is suitable for the material

- = The product is not suitable for the material

M = Pay special attention to the guidelines from the instruments/items manufacturer

This chart is only a guideline and information on the reprocessing of medical devices from the manufacturer according to EN ISO 17664 must always be followed.

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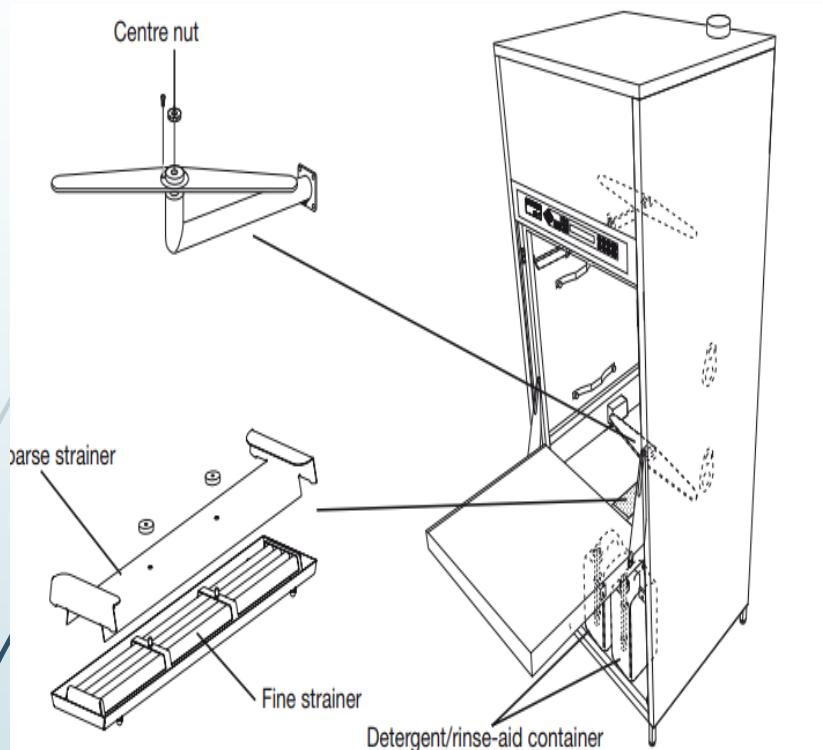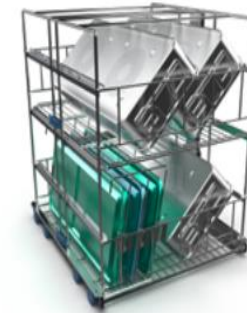
Picture shows the Kit Containers together with the OP wash cart (not incl. in the kit).

Capacity

3 DIN containers 600x300x150 mm incl. lids

Cart for DIN-containers

The container cart is designed to assure efficient cleaning and disinfection. Furthermore, the inclined positioning of the goods ensures maximum run-off and short drying times.

Article No.: 499274000

Capacity

3 x DIN containers 600x300x150 mm incl. lids

Free loading height at bottom level from shelf to spray arm 260 mm, and top level 240 mm

Maximum available shelf space for loading: 592x488 mm

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